

# Health Care Call System

## Pullcord Shower Station Cat. 6158

### Description

Jeron's Cat. 6158 Pullcord Shower Stations are used to signal for assistance from a shower stall or similar location.

- Water-resistant Construction for use in Shower Stalls
- Rugged Stainless Steel Construction
- Reliable Design - No Protruding Parts to Break

### Operation

Pulling the cord from any angle activates all system call signals for an urgent level call. The panel indicator illuminates to verify the call is placed.

Shower calls must be canceled at the originating station by sliding the reset switch upwards.

### Equipment

Pullcord Shower Stations utilize water-resistant construction for use in shower stalls where direct water spray will be encountered.

The station is equipped with a pullcord, reset switch, LED indicator, and engraved 'PULL FOR HELP' label on a stainless steel trimplate for flush wall mount. The panels resist common hospital cleaning agents. Replacement pullcords are available.



When the Pullcord Shower Station is installed with the gaskets for the faceplate and the fasteners, all electronics are sealed behind the panel allowing direct application of water spray from a shower stall or similar location. Tile grout seams may require local application of a sealant.

### Architects/ Engineers Specifications

The Pullcord Shower Station shall be the Jeron Cat. 6158 one-gang with six foot white nylon pullcord, reset button, red call placed LED, and engraved 'PULL FOR HELP' label on a stainless steel faceplate for flush wall mount. It shall withstand direct water spray exposure without wetting of electrical parts or impairing operational specifications in accordance with Sec. 38 'Water Spray Test'. The shower calls shall be reset at the Shower Station only.

### Technical Data

Mounting:  
 Brushed stainless steel 1-gang faceplate with plated steel mounting hardware

Dimensions (H x W x D):  
 4.5 x 2.75 x 2.5 in.  
 (11.4 x 7 x 6.4 cm)

Back box Required:  
 Raco 670RAC 1-gang or equivalent

Controls/Indicators:  
 6 ft. (1.8 m) braided white nylon pullcord with white nylon pendant and metal cord guide with plastic shroud

Slide action reset switch

Red call placed LED indicator

Engraved 'PULL FOR HELP' label with 1/8 in. block characters

Operating Voltage: 28 VDC

Field Connections:  
 Plug-in IDC block; Connector provided

Gaskets:  
 Foam-type for trimplate with knockouts for mounting hardware  
 Rubber "O" ring type (2) for fasteners

Certification:  
 UL 1069 Listed  
 C-UL—Canadian Standard  
 C22.2 NO. 125

Note: For psychiatric application, use Cat. 6146 Pushbutton Bath Station in dry area.

PRODUCT IS SUBJECT TO CHANGE WITHOUT NOTIFICATION