

# Health Care Call System

## Pullcord Bath Station

Cat. 6432

### Description

Jeron Pullcord Bath Stations are used to signal the Nurse Master Station from a bath or similar location.

### Operation

Pulling the cord from any angle activates all system call signals for an urgent level call. The station's LED indicator illuminates to verify that a call has been placed.

Bath calls must be canceled at the originating Bath Station by sliding the reset switch upwards.

### Equipment

Stations are furnished with pull-cord, reset switch, LED call placed indicator and are engraved with a 'PULL FOR HELP' 1/8" high label on a stainless steel trimplate for flush wall mounting. Panels resist common hospital cleaning agents. Replacement pullcords are available.



### Architects/ Engineers Specifications

Pullcord Bath Stations shall be Jeron Cat. 6432 1-gang with six foot white nylon pullcord, reset switch, red call placed LED and engraved 'PULL FOR HELP' label on a stainless steel faceplate for flush wall mounting.

Bath calls shall be canceled at the originating Bath Station only.

### Technical Data

Mounting: Brushed stainless steel 1-gang faceplate with plated steel mounting hardware

Dimensions (H x W x D):  
4.5 x 2.75 x 2.5 in.  
(11.4 x 7 x 6.4 cm)

Back box Required: Raco 670RAC 1-gang mounted 1/2" behind wall face or Raco 232 2-gang with Raco 772 1-gang raised cover

Controls/Indicators:  
Pullcord: 6 ft. (1.8 m) braided, white, nylon cord with white nylon pendant and metal cord guide with plastic shroud  
Reset Switch: Slide action  
Call Placed Indicator: Red LED

Operating Voltage: 28 VDC

Field Connections:  
Plug-in IDC block

Certification:  
UL 1069 Listed  
C-UL—Canadian Standard  
C22.2 NO.125

Note: For psychiatric application, use Cat. 6446 Pushbutton Bath Station.

PRODUCT SUBJECT TO CHANGE WITHOUT NOTIFICATION