



Description

The Jeron Cat. 8710 Staff Station is used in EC-300 Tone-Visual Nurse Call Systems to activate call signals from staff locations.

- Long-life LED Indicator
- No special tools or connectors required

Operation

When the PUSH FOR HELP button is pressed, the station's CALL PLACED indicator illuminates, the dome and zone lamps illuminate and corresponding room and call placed indicators illuminate at the master annunciator accompanied by an audible alert tone. The calls must be canceled by pushing the CANCEL button at the originating station.

Equipment

The staff station is equipped with CALL PLACED INDICATOR, PUSH FOR HELP and CANCEL pushbuttons. It is equipped with a brushed stainless steel faceplate for flush wall mount in the backbox. The wire leads are color coded with striped ends. The station withstands common hospital cleaning agents.



Architects/ Engineers Specifications

The Staff Station shall be the Jeron Cat. 8710 equipped with a CALL PLACED LED, PUSH FOR HELP and CANCEL pushbuttons. The station shall have a brushed stainless steel faceplate for flush wall mounting on a 1-gang backbox.

Technical Data

Mounting:

Brushed stainless steel 1-gang faceplate with stainless steel mounting hardware

Dimensions (H x W x D):

4.5 x 2.75 x 1.75 in.
(11.4 x 7 x 4.5 cm)

Backbox Required:

Raco 670RAC 1-gang or equivalent

Controls/Indicators:

Red CALL PLACED indicator
PUSH FOR HELP pushbutton
CANCEL pushbutton

Operating Voltage:

28 VDC

Field Connections:

8 in. (20 cm) color coded wire leads with stripped ends and wire nuts

Certification:

UL 1069 Listed
C-UL—Canadian Standard
C22.2 NO.125

PRODUCT IS SUBJECT TO CHANGE WITHOUT NOTIFICATION