



Health Care Call System

Patient Stations Cat. 8724/8725

Description

The Jeron Cat. 8724 (pictured) and 8725 Patient Stations are used in the EC-300 Tone-Visual Nurse Call Systems to signal for assistance and/or control entertainment audio. They may be used with either the Cat. 6838 Pillow Speaker or the Cat. 6833/6835 Callcords.

- Molded High-Impact Plastic
- Long-Life LED Indicator
- No Special Tools or Connectors Required

Operation

When the CALL button on a callcord is pressed, the station's CALL PLACED indicator illuminates. If so equipped, the dome and zone lamps will light. At the master annunciator, the corresponding room and call placed indicators illuminate, accompanied by an audible alert tone. Calls must be canceled by pushing the CANCEL button at the originating station.

The removal of a callcord or a pillow speaker places a normal call on the system that can only be canceled by reinserting the cord and pressing the CANCEL button. A Cat. 6836 Dummy Plug may be inserted into the jack to prevent cordout calls.



Equipment

The Patient Stations are equipped with a call placed indicator, callcord jack (two for Cat. 8725), momentary contact CANCEL pushbutton and white plastic faceplate for flush wall mount. The panels withstand common hospital cleaning agents.

Architects/ Engineers Specifications

The Patient Station shall be Jeron Cat. 8724/8725 equipped with a CALL PLACED LED, CANCEL pushbutton and DIN callcord jack(s) accepting either the Cat. 6838 Pillow Speaker or the Cat. 6833/6835 Callcord. The station shall have a white plastic faceplate for flush wall mounting on a 1-gang backbox.

Patient stations that do not allow the exchange of pillow speaker and callcord are not acceptable.

Technical Data

Mounting:

White thermoset plastic 1-gang trimplate and matching hardware

Dimensions (H x W x D):

4.5 x 2.75 x 2.5 in. (11.4 x 7 x 6.4 cm)

Backbox Required:

Raco 670RAC 1-gang or equivalent mounted 1/2" behind wall face or Raco 232 or equivalent with Raco 722 raised cover

Controls/Indicators:

Red CALL PLACED indicator

CANCEL pushbutton

DIN callcord jack (two for Cat. 8725)

Operating Voltage: 28 VDC

Field Connections:

8 in. (20 cm) color coded wire leads with stripped ends and wire nuts

Certification:

UL 1069 Listed

C-UL—Canadian Standard

C22.2 NO.125

Associated Jeron Equipment

Choose from the following:

- Cat. 6833/6835 Callcords
- Cat. 6879 Geriatric Callcord
- Cat. 6836 Dummy Plug
- Cat. 6838 Pillow Speaker

PRODUCT IS SUBJECT TO CHANGE WITHOUT NOTIFICATION