


Technical Data

Emergency Station (Pull Cord) SERIES 9100, 9600 Patient Station (Pull Cord Type) SERIES 9100

FEATURES:


- Six Foot Pull Cord and Pendant with Breakaway Connector
- Red "Call-Placed" Light
- Faceplate Engraved "Pull For Help"
- Single Gang Mounting

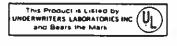

FUNCTION:

The Auth Emergency/Patient Station provides multiple use as an emergency station in the Auth 9600 SERIES Audio Visual Nurse Call System and as an emergency station and/or patient station in the Auth 9100 SERIES Visual Nurse Call System.

cated in bath and toilet areas when used as an Emergency ation unit provides emergency call placement with reset capability attainable only at the call-initiating station. When used as a Patient Station, unit provides either normal call placement or emergency call placement with reset capability attainable only at the call-initiating station.

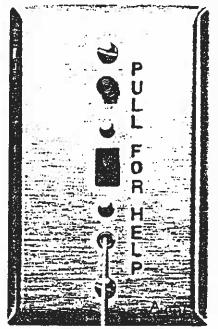

ENGINEERS' SPECIFICATIONS

The Emergency and/or Patient Station shall be Auth Catalog No. 945300-00 as indicated on plans.

- The Emergency and/or Patient Station shall consist of a mechanically locking type switch. Switch shall be connected to six (6) foot pullcord and pendant by a breakaway connector. Screw type terminals shall permit rapid field installation.
- 2. All components shall be mounted to a satin stainless steel faceplate. The faceplate shall be engraved "PULL FOR HELP."
- Emergency and/or Patient Station shall be equipped with a red "call placed" light to assure the patient his call has been placed.
- 4. It shall be possible at the time of installation of the Patient Station to change call placement from normal

to emergency without the change or addition of any components.

- Throughout the system initiation of an emergency call shall override normal calls without cancelling them.
- 6. Emergency and/or Patient Station shall be mountable on a single gang electrical backbox. (Steel City #58371, Appleton #4SSL or equal). Surface mounting requirements shall be wiremold device No. 5744S or equal. Minimum backbox depth shall be 2". Box shall be furnished by the installer.
- Emergency Station and/or Patient Station shall be Underwriters Laboratories Inc. listed (UL 1069).



