



# Model No. 4T-2B

## Dual Patient Station

### DESCRIPTION:

The 4T-2B is a patient bedside station that is intended to be wall mounted at the head wall between two patient beds. The station includes two call cord receptacles which will accept a variety of 1/4" jack momentary call cord sets. The station also includes a room indicator LED to show the presence of an active call. The circuit includes a single enunciator point to identify by room at the nurse station.

When a call cord is pressed the call will illuminate in three locations simultaneously. The beds' call assurance indicator, the rooms' corridor dome light, and the enunciator panel at the nurses' station. In addition, a slow repeating call tone will sound at the nurses' station and any associated duty stations.

Removal of either call cord set will automatically initiate a call. The call cord or a dummy plug must be replaced before the call can be reset.

The station can only be reset at the point of origin unless an in-room remote reset station has been added for your convenience.

### OPTIONAL ACCESSORIES:

CS-1 Momentary Cord set

DP-1 Dummy Plug

4T-RR Remote Reset station

B09050-04 Connector pigtail



### FEATURES:

- Durable faceplate
- Single room indicator
- Plug-in wiring header for easy removal
- Scratch resistant Lexan® overlay
- Cord disengaged safety feature

### SPECIFICATIONS:

**Circuit Board:** PCB mounted header for use with insulation displacement connectors. Note: Connectors sold separately.

**Back Box:** Single gang 2-1/2" deep. Steel City #CD or equal.

**Dimensions:** 4.50"H x 2.75" W x 2.00"D

**Faceplate:** Constructed of UL94V-0 ABS flame resistant plastic.

**Switch:** Momentary activation, mechanical life >1 million operations.

**Hardware:** Includes stainless steel mounting screws.